

## CLINICIAN QUICK GUIDE FOR DELAYED START PROCEDURE

This reference will help you understand the Delayed Start settings of the Curlin Medical Infusion Pumps (Not available in PCA mode). For more detailed information, please refer to the User's Manual issued with the Pump.

1. Be sure that the pump displays the correct date and time on the welcome screen:

```

WELCOME
27JAN03 17:24
Rev :RSH-F4-B0
Self Test.
```

2. At the "Delay" field in the initial pre-prescription menu, the default setting of OFF will be highlighted and blinking. To enter a delayed start time, press the NO/CHANGE key to change the setting from OFF to ON and then press the YES key.

```

Z000 UNITS:ML
      DELAY:OFF
      Titrate:OFF
      NEXT? YES
```

3. A screen will display and prompt you to enter the actual time (HH:MM) and date (mm/dd/yy) that the therapy should start. Note: The time format (12 hr AM/PM or 24 hr Military) selected in the BIOMED SETUP Menu (DATE AND TIME) will apply in this table; and, if 24 hr Military was selected, the AM/PM field will not appear here. When these fields are correct, press YES at the "Accept?" field.

<pre style="border: 1px solid black; padding: 5px;"> Z000 DELAY START       TIME: 0:00 HH:MM       DATE: mm/dd/yy       ACCEPT? YES/NO</pre>	<pre style="border: 1px solid black; padding: 5px;"> Z000 DELAY START       AM/PM: AM       TIME: 0:00 HH:MM       DATE: mm/dd/yy       ACCEPT? YES/NO</pre>
--	--

4. Continue entering the rest of the prescription as prompted.
5. Remember the KVO Rate you program will be the KVO Rate during the "Delayed Start" period.
6. When programming the prescription is complete, the following screen will display.

```

PRESS
RUN to START
NO to REVIEW
OPTIONS or PRIME
```

7. The pump can now be turned OFF and sent out to the patient's home. The "Delayed Start" program will be retained.


8. When the pump is turned on again (at the patient's home, for instance), the patient or nurse should install the administration set, press YES at the **PROGRAM** field and then press YES at the **REPEAT Rx** field.



9. The cursor will automatically advance through the prescription menus. You can stop the auto-advance function by pressing any key (except ON/OFF) and then complete the confirmation of the prescription manually by pressing the YES key at the remaining fields.

**Note: If more than one therapy is programmed, a “Select Therapy Screen” will list all preprogrammed therapies. PLEASE ENSURE THAT THERE ARE NO UN-INTENDED THERAPIES.**

10. At the end of the scroll, the pump will display the following screen:

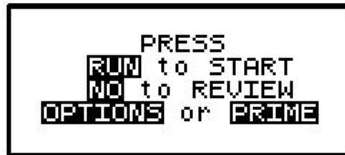

11. If the nurse or patient wants to review or change the prescription, they can press the **NO** key to review the program, and confirm or change the prescription. When the prescription is confirmed, the above screen will display again.
12. Pressing the **RUN** key now will display the Delayed Start screen and continue to count down until the infusion begins.

