

## PATIENT INFORMATION GUIDE

### Intermittent Therapy —Curlin 6000™ V2.01 CMS Infusion Pump



This reference will help you remember the steps your clinician has shown you about infusing your therapy using the Curlin 6000 CMS Pump.

1. Turn the pump on by pressing the orange **ON/OFF** key.

2. The pump will run through a series of self tests. One of the screens will display a bar that represents the power left in the batteries. (Replace batteries with Duracell ProCell C-cell batteries when 1/3 or less of the bar remains).

3. Highlight **PROGRAM** using the up or down arrow keys. Press the **YES** key.

**NOTE:** Program mode may or may not be enabled.

**NOTE:** Pressing the **YES** key selects and confirms the highlighted line.

ENTER TO SELECT
<b>PROGRAM</b>
LIBRARY

**NOTE:** If instructed to use a medication in the Protocol Library, select **LIBRARY**. Select **REPEAT Rx** then press the **YES** key.

The next screen may look like one of the following screens:

### REPEAT Rx

(To hang a new bag with the same prescription or to begin the initial bag.)

Press **YES** to **REPEAT Rx**.

ENTER TO SELECT
<b>REPEAT Rx</b>
NEW PROGRAM

When you press the **YES** key to select **REPEAT Rx**, the cursor will automatically advance through the lines. You can stop the scrolling and revert to manual by pressing any key (except **ON/OFF**), then manually confirm each field by pressing the **YES** key at every line.

I N T	UNITS:
	CONCEN:
	DELAY:
	NEXT? YES

I N T	BAG Vol:
	AMT/DOSE:
	DOSE RATE:
	DOSE TIME:
	DOSE Freq:
	KVO Rate:
	DOSE/BAG
	REQ Vol:
TOT Time:	
DONE? YES	

I N T	NEXT DOSE:
	DUE IN [ ] HH:MM
	DONE? YES

The last screen of the program indicates how long until the next dose starts delivering. Be sure you are connected to the pump prior to this time.

When the following screen appears, press the **RUN** key.

<b>PRESS</b>
<b>RUN to START</b>
<b>NO to REVIEW</b>
<b>OPTIONS or PRIME</b>

**NOTE:** Selecting **REPEAT Rx** while a dose is in progress will drop that dose. Therefore hang the new bag prior to the dose's delivery time.

### RESUME

(To restart infusion that is not finished)  
Press **YES** to **RESUME**.

<b>ENTER TO SELECT</b>
<b>RESUME</b>
<b>REPEAT Rx</b>
<b>NEW PROGRAM</b>

(**NOTE:** Do not arrow down to **NEW PROGRAM**.  
If selected, turn pump off and on again.)

PG. 5

If you selected **RESUME**, the following screen will appear. Press the **RUN** key.

<b>PRESS</b>
<b>RUN to START</b>
<b>NO to REVIEW</b>
<b>OPTIONS or PRIME</b>

Whenever your therapy is infusing, the following screen displays and the green status light blinks.

I N T	<b>RATE or DOSE # of #</b>
	[.....]
	<b>Dose INF: or TOT Vol</b>
	<b>REMAIN: ml or HH:MM</b>

If a delayed start is programmed, the following screen will display:

<b>DELAYED START</b>
<b>INFUSION WILL</b>
<b>START IN: HH:MM</b>
<b>KVO RATE: mL/HR</b>

PG. 6

**NOTE:** If the pump is turned off while a dose is in progress, turn the pump back on within 30 minutes to maintain the same schedule.

**Tip:** Green light blinks when the pump is running.

Yellow light blinks when the pump is paused.

Red light blinks when there is an alarm.

PG. 7

To resolve the alarm: Read the alarm, press the **PAUSE** key, resolve the alarm, then press **YES** to **RESUME** and **RUN** to start.



Moog Medical Devices Group  
4314 Zevex Park Lane  
Salt Lake City, Utah 84123 USA

tf 800.970.2337  
ph 801.264.1001  
fx 801.264.1051

[www.moog.com/medical](http://www.moog.com/medical)

Moog is a registered trademark of Moog, Inc. and its subsidiaries. All trademarks indicated herein are the property of Moog, Inc. and its subsidiaries.  
© 2011 Moog, Inc. All rights reserved.  
46361 Rev. A

PG. 8

Patient Name \_\_\_\_\_

Date \_\_\_\_\_

Contact Information \_\_\_\_\_

Special Instructions \_\_\_\_\_