

PATIENT INFORMATION GUIDE

TPN Therapy —Curlin 6000™ V2.01 CMS Infusion Pump



This reference will help you remember the steps your clinician has shown you about infusing your therapy using the Curlin 6000 CMS Pump.

1. Turn the pump on by pressing the orange **ON/OFF** key.

2. The pump will run through a series of self tests. One of the screens will display a bar that represents the power left in the batteries. (Replace batteries with Duracell ProCell C-cell batteries when 1/3 or less of the bar remains).

3. Highlight **PROGRAM** using the up or down arrow keys. Press the **YES** key.

NOTE: Program mode may or may not be enabled.

NOTE: Pressing the **YES** key selects and confirms the highlighted line.

ENTER TO SELECT
PROGRAM
LIBRARY

NOTE: If instructed to use a medication in the Protocol Library, select **LIBRARY**. Select **REPEAT Rx** then press the **YES** key.

The next screen may look like one of the following screens:

REPEAT Rx

(To hang a new bag with the same prescription or to begin the initial bag.)

Press **YES** to **REPEAT Rx**.

ENTER TO SELECT
REPEAT Rx
NEW PROGRAM

After pressing the **YES** key to select **REPEAT Rx**, TPN mode will be highlighted. Press the **YES** key.

The cursor will then automatically advance through the lines. You can stop the scrolling and revert to manual by pressing any key (except **ON/OFF**), then manually confirm each field by pressing the **YES** key at every line.

TPN	DELAY: OFF
	NEXT? YES

TPN	BAG Vol:
	AMT TBI:
	INF RATE:
	UP Ramp:
	DN Ramp:
	TOT Time:
	KVO Rate:
	DONE? YES

When the following screen appears, press the **RUN** key.

PRESS
RUN to START
NO to REVIEW
OPTIONS or PRIME

RESUME

(To restart infusion that is not finished)
Press **YES** to **RESUME**.

ENTER TO SELECT
RESUME
REPEAT Rx
NEW PROGRAM

(NOTE: Do not arrow down to **NEW PROGRAM**.
If selected, turn pump off and on again.)

If you selected **RESUME**, the following screen will appear. Press the **RUN** key.

PRESS
RUN to START
NO to REVIEW
OPTIONS or PRIME

PG. 5

Whenever your therapy is infusing, the following screen displays and the green status light blinks.

T P N	RATE
	[.....]
	TOTAL VOL:
	REMAIN:

If a delayed start is programmed, the following screen will display:

DELAYED START
INFUSION WILL
START IN: HH:MM
KVO RATE: mL/HR

NOTE: In an emergency, if an early down ramp is necessary, press the **HELP/OPTIONS** key while the pump is running. Arrow down to “**DN Ramp**” and press **NO** to change **OFF** to **ON**, then press **YES**. Enter the amount of time the TPN

PG. 6

should taper down over or accept the given entry. Press **YES** after the number of hours and minutes entered, then press **YES** to Accept, then **YES** to Accept Options? The pump will begin to taper the rate down immediately.

Tip: Green light blinks when the pump is running.

Yellow light blinks when the pump is paused.

Red light blinks when there is an alarm.

PG. 7

To resolve the alarm: Read the alarm, press the **PAUSE** key, resolve the alarm, then press **YES** to **RESUME** and **RUN** to start.

MOOG

Moog Medical Devices Group
4314 Zevex Park Lane
Salt Lake City, Utah 84123 USA

tf 800.970.2337
ph 801.264.1001
fx 801.264.1051

www.moog.com/medical

Moog is a registered trademark of Moog, Inc. and its subsidiaries. All trademarks indicated herein are the property of Moog, Inc. and its subsidiaries.
© 2011 Moog, Inc. All rights reserved.
46365 Rev. A

PG. 8

Patient Name _____

Date _____

Contact Information _____

Special Instructions _____