

# PATIENT QUICK GUIDE FOR CONTINUOUS THERAPY

350-9344C

This reference will help you remember the steps your clinician has shown you about infusing your therapy using the Curlin Medical Infusion Pumps.

1. Turn the pump on by pressing the blue ON/OFF key
2. A series of screens will automatically appear. One of the screens will display a bar that represents the power left in the batteries (replace when ¼ of bar or less remains)
3. Screens which require you to press the **YES** key are shown below in sequence:

The screen will have a black bar or cursor on one of the rows of text. To move the cursor from one field to another, use the blue up and down arrow keys.

The first screen has a cursor on the "PROGRAM" field. Press the **YES** key.




The second screen may look like either of the ones below  
(DO NOT MOVE CURSOR TO NEW PROGRAM)


**RESUME** (To restart infusion that is not finished)  
Press YES



If you selected RESUME, the following screen will appear. Press RUN key

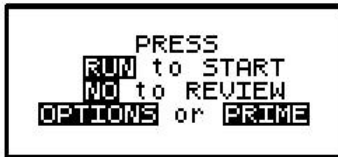

Whenever your therapy is infusing, a number of information screens (such as the following) will display and the green status light blinks

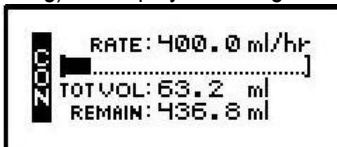




When a delay start is programmed, the following screen will display




**REPEAT Rx** (To hang new bag with same prescription)  
Press YES



When you press the YES key to select REPEAT Rx, the following screens will appear. For all pumps, except **2000 Plus™** using Pre-R39 software, the cursor will automatically advance through the following screens. To stop the auto-advance press any key, except ON/OFF, and then complete the confirmation of the prescription manually by pressing the YES key at each remaining field.

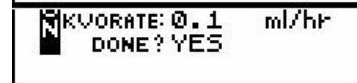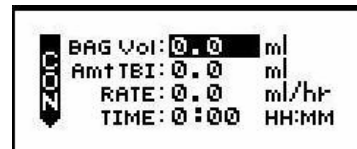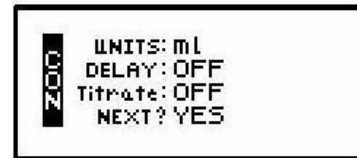

When the following screen appears, press RUN key

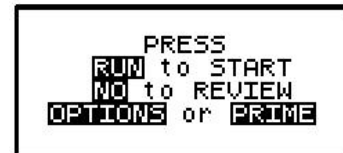

**Tip:** Green light blinks when the pump is running.  
Yellow light blinks when the pump is paused.  
Red light blinks when there is an alarm.

Patient Name \_\_\_\_\_

Date \_\_\_\_\_

Special Instructions \_\_\_\_\_