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


CURLIN MEDICAL, LLC
WORKSHEET FOR PROGRAMMING
CONTINUOUS MODE



PATIENT: _____

PROGRAMMING / SETUP SEQUENCE

Turn the Pump on, watch self test and power source bar. Sets must be installed for pump to run.

Press the **↑** and **↓** keys to scroll through the menu, and press the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

- 1 Press **YES** to select **PROGRAM**
- 2 Highlight **NEW PROGRAM**, then press **YES**
- 3 Press **YES** to select **CONTINUOUS**
- 4 **UNITS:** Press **YES** to select **ml** (or **NO** to change to **mg** or **mcg**), then press **YES**

CONCEN: If **mg** or **mcg** selected, enter **CONCENTRATION**, then press **YES**

- 5 **DELAY:** Press **YES** to select **OFF** (or **NO** to change to **ON**), then press **YES**

- 6 **Titrate:** Press **YES** to select **OFF**

- 7 Press **YES** to select **NEXT? YES**

8 **BAG Vol:** Enter **BAG VOLUME**, press **YES** **BAG Vol...**

9 **Amt TBI:** Enter the **AMOUNT TO BE INFUSED**, press **YES** **AMT TBI...**

10 **RATE:** Enter **RATE**, press **YES** **RATE.....**

- 11 **TIME:** Is automatically calculated if the rate has been entered, press **YES**

12 **KVO RATE:** Enter **KVO RATE**, then press **YES** (0 to 10 ml/hr) **KVO RATE.**

- 13 **DONE ?:** Press **YES** to accept

- 14 **OPTIONS:** Pressing the **HELP/OPTIONS** key will allow you to set the Air-In-Line, Down Occlusion setting of High or Low and the Lock level. Press **YES** to accept all the new options.

- 15 **RUN to START:** Press **RUN** to begin infusion.

Programmed By _____

Date _____

Checked By _____

Date _____