



**CURLIN MEDICAL, LLC**  
**WORKSHEET FOR PROGRAMMING**  
**INTERMITTENT MODE**

PATIENT: \_\_\_\_\_

**PROGRAMMING / SETUP SEQUENCE**

Turn the Pump on, watch self test and power source bar. Sets must be installed for pump to run.

Press the  and  keys to scroll through the menu, and press the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

- 1 Press **YES** to select **PROGRAM**
- 2 Highlight **NEW PROGRAM**, then press **YES**
- 3 Select **INTERMITTENT** and press **YES**
- 4 **UNITS:** Press **YES** to select **ml** (or **NO** to change to **mg** or **mcg**), then press **YES**  
**CONCEN:** If **mg** or **mcg** selected, enter **CONCENTRATION**, then press **YES**
- 5 **DELAY:** Press **YES** to select **OFF** (or **NO** to change to **ON**), then press **YES**
- 6 Press **YES** to select **NEXT? YES**
- 7 **BAG Vol:** Enter **BAG VOLUME**, press **YES** **BAG Vol.....**
- 8 **AMT/DOSE:** Enter **AMOUNT**, press **YES** **AMT/DOSE...**
- 9 **DOSE RATE:** Enter **RATE**, press **YES** **DOSE RATE.**
- 10 **DOSE Time:** Enter number of **Minutes** per **DOSE**, press **YES** **DOSE Time.**
- 11 **DOSE Freq:** Enter **Hours** between **DOSES**, press **YES** **DOSE Freq.**
- 12 **KVO RATE:** Enter **KVO RATE**, then press **YES** (0 to 10 ml/hr) **KVO RATE.**
- 13 **DOSE/BAG:** Auto calculated, press **NO** to change **DOSE/Bag...**
- 14 **REQ Vol:** Auto calculated, press **YES** (no change allowed) **REQ Vol...**
- 15 **TOT Time:** Auto calculated, press **YES** (no change allowed) **TOT Time..**
- 16 **DONE ?:** Press **YES** to accept
- 17 **OPTIONS:** Pressing the **HELP/OPTIONS** key will allow you to set the Air-In-Line, Down Occlusion setting of High or Low and the Lock level. Press **YES** to accept all the new options.
- 18 **RUN to START:** Press **RUN** to begin infusion.

Programmed By \_\_\_\_\_

Date \_\_\_\_\_

Checked By \_\_\_\_\_

Date \_\_\_\_\_