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




CURLIN MEDICAL, LLC
WORKSHEET FOR PROGRAMMING
PCA MODE



PATIENT: _____

PROGRAMMING / SETUP SEQUENCE

Turn the Pump on, watch self test and power source bar. Sets must be installed for pump to run.

Press the  and  keys to scroll through the menu, and press the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

1 Press **YES** to select **PROGRAM**

2 Highlight **NEW PROGRAM**, then press **YES**

3 Select PCA and Press **YES**

4 **UNITS:** Press **YES** to select **mg** (or **NO** to change to **mcg** or **ml**), then press **YES**

CONCEN: If **mg** or **mcg** selected, enter **CONCENTRATION**, then press **YES**

5 **Admin Route:** Press **YES** to select **IV** (or **NO** to change to **EPI** or **SQ**), then press **YES**

6 **Load Dose:** Enter **AMOUNT**, then press **YES** **Load Dose..**

7 **Titrate:** Press **YES** to select **OFF**

8 Press **YES** to select **NEXT? YES**

9 **BAG Vol:** Enter **BAG VOLUME**, press **YES** **BAG Vol.....**

10 **Basl Rate:** Enter **RATE**, press **YES** **Basal Rate..**

11 **Bolus:** Enter **AMOUNT**, press **YES** **Bolus.....**

12 **Bolus Interval:** Enter **Minutes** between **Bolus**, press **YES** **Bolus Interval.**

13 **# Bols/hr:** Enter number of **Boluses** per **Hour**, press **YES** **# Bolus/HR..**

14 **DONE ?:** Press **YES** to accept

15 **OPTIONS:** Pressing the **HELP/OPTIONS** key will allow you to set the Air-In-Line, Down Occlusion setting of High or Low and the Lock level. Press **YES** to accept all the new options.

16 **RUN to START:** Press **RUN** to begin infusion.

Programmed By _____

Date _____

Checked By _____

Date _____