



CURLIN MEDICAL, LLC
WORKSHEET FOR PROGRAMMING
TPN MODE

PATIENT: _____

PROGRAMMING / SETUP SEQUENCE

Turn the Pump on, watch self test and power source bar. Sets must be installed for pump to run.

Press the  and  keys to scroll through the menu, and press the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

- 1 Press **YES** to select **PROGRAM**
- 2 Highlight **NEW PROGRAM**, then press **YES**
- 3 Select **TPN** and press **YES**
- 4 **DELAY**: Press **YES** to select **OFF** (or **NO** to change to **ON**), then press **YES**
- 5 Press **YES** to select **NEXT? YES**
- 6 **BAG Vol**: Enter **BAG VOLUME**, press **YES** **BAG Vol...**
- 7 **Vol TBI**: Enter **VOLUME TO BE INFUSED**, press **YES** **Vol TBI...**
- 8 **Inf RATE**: Enter the **INFUSION RATE** or skip and enter the next three fields. When entered, press **YES** **Inf RATE.**
- 9 **UP Ramp**: Enter **Up Ramp** time in **HH:MM**, press **YES** **Up Ramp...**
- 10 **DN Ramp**: Enter **Down Ramp** time in **HH:MM**, press **YES** **Down Ramp.**
- 11 **TOT Time**: Enter **Total Time** for infusion in **HH:MM**, press **YES** **TOT Time...**
- 12 **KVO RATE**: Enter **KVO RATE**, then press **YES** (0 to 10 ml/hr) **KVO RATE.**
- 13 **DONE ?**: Press **YES** to accept
- 14 **OPTIONS**: Pressing the **HELP/OPTIONS** key will allow you to set the Air-In-Line, Down Occlusion setting of High or Low and the Lock level. Press **YES** to accept all the new options.
- 15 **RUN to START**: Press **RUN** to begin infusion.

Programmed By _____

Date _____

Checked By _____

Date _____