



CURLIN MEDICAL, LLC
WORKSHEET FOR PROGRAMMING
VARIABLE MODE

PATIENT: _____

PROGRAMMING / SETUP SEQUENCE

Turn the Pump on, watch self test and power source bar. Sets must be installed for pump to run.

Press the  and  keys to scroll through the menu, and press the **YES** and **NO** keys to accept/confirm or reject/change a field entry.

1 Press **YES** to select **PROGRAM**

2 Highlight **NEW PROGRAM**, then press **YES**

3 Select **VAR** and press **YES**

4 **UNITS:** Press **YES** to select **ml** (or **NO** to change to **mg** or **mcg**), then press **YES**

CONCEN: If **mg** or **mcg** selected, enter **CONCENTRATION**, then press **YES**

5 **DELAY:** Press **YES** to select **OFF** (or **NO** to change to **ON**), then press **YES**

6 Press **YES** to select **NEXT? YES**

7 **BAG Vol:** Enter **BAG VOLUME**, press **YES**

BAG Vol....

8 **# DOSES:** Enter the **TOTAL NUMBER OF DOSES**, press **YES** *** **# DOSES...**

9 **KVO RATE:** Enter **KVO RATE**, press **YES** (0 to 10 ml/hr)

KVO RATE.

10 Press **YES** to select **NEXT? YES**

DOSE: 1 OF # (Example of Automatically Calculating **TIME**)

11 **Amt TBI:** Enter the **AMOUNT TO BE INFUSED**, press **YES**

AMT TBI....

12 **RATE:** Enter **RATE**, press **YES**

RATE.....

13 **TIME:** Is automatically calculated if the Amt TBI and rate have been entered, press **YES**

14 Press **YES** to select **NEXT? YES**

DOSE: 2 OF # (Example of Automatically Calculating Amt TBI)

15 **Amt TBI:** Press ↓ to skip the **AMOUNT TO BE INFUSED**

16 **RATE:** Enter **RATE**, press **YES**

RATE.....

17 **Time:** Enter **Time** for infusion in **HH:MM**, press **YES**

TIME.....

18 **Amt TBI:** Is automatically calculated if the rate and time have been entered, press **YES**

19 **RATE:** Press ↓ to skip the **RATE**

20 **TIME:** Press ↓ to skip the **TIME**

21 Press **YES** to select **NEXT? YES**

DOSE: 3 OF # (Example of Automatically Calculating RATE)

22 **Amt TBI:** Enter the **AMOUNT TO BE INFUSED**, press **YES**

AMT TBI...

23 **RATE:** Press ↓ to skip the **RATE**

24 **Time:** Enter **Time** for infusion in **HH:MM**, press **YES**

TIME.....

25 **RATE:** Is automatically calculated if the Amt TBI and time have been entered, press **YES**

26 **TIME:** Press ↓ to skip the **TIME**

27 Press **YES** to select **NEXT? YES**

28 A "Summary Table" displaying **TOTAL DOSES**; **TOTAL TIME** and **REQUIRED VOLUME** will be displayed. No changes are allowed at this point

29 **DONE ?:** Press **YES** to accept

30 **OPTIONS:** Pressing the **HELP/OPTIONS** key will allow you to set the Air-In-Line, Down Occlusion setting of High or Low and the Lock level. Press **YES** to accept all the new options.

31 **RUN to START:** Press **RUN** to begin infusion.

*** A total of up to 24 doses may be programmed. Three (3) doses are displayed here, each showing a different variable to be calculated.

Programmed By _____

Date _____

Checked By _____

Date _____